Avicenum **ORTHO 1500**

(EN) INSTRUCTION MANUAL

Avicenum ORTHO 1500 Knee Orthosis Type 02

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Avicenum ORTHO 1500 bandáž kolenná typ 02

(EN) Dear customer,

in your hands, you are now holding a knee orthosis for post-operative and post-traumatic therapy. You can maximize the benefits of the product by its proper use and maintenance.

Avicenum ORTHO 1500 Knee Orthosis Type 02 is manufactured by the flat knitting technology. The materials used are non-irritating and latex-free. The product is reinforced with flat spiral stainless steel springs on the sides. In the patellar part, the product is completed with a circular silicone pad that slightly compresses the patella and ensures its correct position in the knee joint during movement. In the upper and lower part, the product is supplemented with velcro straps (preventing its movement) and, on the inner side, it is fitted with anti-slip strips with silicone studs along the entire length of the flat spiral springs. The mechanism behind the effects of the product is based on the compressive action in the knee area which produces the desired therapeutic effect. In order to achieve the desired effects, you must select the correct size of the product. The product is most effective during physical strain. The product is also suitable for sports activities. You should remove the product during longer breaks. **Avicenum ORTHO 1500 Knee Orthosis Type 02** is a medical device intended for use both at home and in medical facilities.

Product name:

Avicenum ORTHO 1500 Knee Orthosis Type 02

Product variants:

- Double-sided, straps, silicone pad, sizes XS – 5XL

Intended use:

A medical device designed to support the knee during post-traumatic and post-operative therapy, in case of flabby muscles, inflammation, or injury to knee ligaments and tendons and to prevent swelling. This product is also suitable for sports activities as a therapeutic or preventive aid.

Indications:

- Acute or chronic knee instability
- Treatment after severe or mild distortions of the knee (sprain)
- Knee post-traumatic swelling
- Patella chondropathy
- Knee swelling in arthritis or arthrosis
- Treatment after surgery or fracture of the knee
- Painful syndromes of the knee
- Prevention of knee injury
- Knee ligaments instability

If this is the first time you use the product, consult your doctor or have your medical condition regularly examined.

Contraindications:

- Skin diseases or injuries in the knee area
- Acute weeping skin manifestations
- Hypersensitivity to materials used
- Lymphatic drainage disorders and unspecified swelling of the soft tissue in the knee area
- Limb sensitivity disorders, peripheral neuropathy (e.g., in diabetes mellitus or varicose veins)

Your doctor should consider the use of the product in view of the known contraindications and possible risks compared to the product's therapeutic benefits.

Target group:

Patients with knee instability, patients undergoing post-operative or post-traumatic therapy of the knee, and patients with the above indications. The product is intended for use on healthy, intact skin. The product is intended for all age groups and both sexes. The product may be used in pregnancy without limitation.

Side effects:

Side effects may occur especially when using a wrong size of the product or if the product is used outside strenuous activities. If the product is too tight, there may be pressure sores, impaired blood circulation due to compression of blood vessels, or deterioration of sensitivity due to excessive compression of the nerves. Sensitive and hypersensitive individuals may experience skin irritation (redness, itching, or blistering) at the point of contact of the product with the skin.

How to choose the correct size:

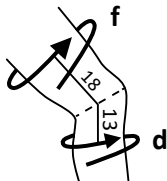
In order to achieve the desired effects, you must select the correct size of the product. Determine the size by measuring the circumference at the points indicated in the figure.

Note:

Any swelling or instability of the knee should be examined by your doctor. If this is the first time you experience swelling or instability of the knee, consult your doctor before using the product. Do not use the product if you do not know the cause of the swelling or instability of the knee. Always consult your doctor before combining the product with other medical devices (e.g., compression stockings).

- Use a tape measure
- Take the measurements standing up, preferably in the morning right after waking up
- Get assistance from another person
- Measured values may change during the therapy – always verify the correct size before using a new product

	Circumference 18 cm above the centre of the knee joint (cm)	Circumference 13 cm below the centre of the knee joint (cm)
XS	38 - 41	28 - 31
S	41 - 44	31 - 34
M	44 - 47	34 - 37
L	47 - 50	37 - 40
XL	50 - 53	40 - 43
XXL	53 - 56	43 - 46
3XL	56 - 59	46 - 49
4XL	59 - 62	49 - 52
5XL	62 - 65	52 - 55

If you are on the border between two sizes, choose the bigger one. If you have difficulties choosing the size of the product, visit a specialized medical supplies store or directly the manufacturer.

If you have difficulties putting on the product, visit a specialized medical supplies store and request a demonstration of the correct method of applying it.

With a correctly selected size:

- You should feel noticeable but not unpleasant pressure at the knee area
- The product does not slide or roll down when walking
- The product does not choke at any point

Size and manufacturer information can be found on the tag sewn at the top edge of the product.

Method of use:

Grab the product by the side spiral supports in their upper part and then gradually pull the product over the knee so that the circular silicone pad fits to the patella. Never use force. It is best to put on the product in the morning (while sitting) before any physical strain. When putting on, the product may not come into contact with sharp objects (e.g., jewellery) or long nails.

Stretch the product evenly without creases for effective distribution of pressure. Finish by tightening the velcro straps so that you feel mild pressure.

If you use body cosmetics, wait about 15 minutes after application before you put on the product. In the case of visible mechanical damage to the product, stop wearing it and preferably replace it with a new product.

Care and maintenance:

Avicenum ORTHO 1500 Knee Orthosis Type 02 is a medical device intended for repeated use by a single person. The product will maintain its properties and safety for 12 months from the first use under these conditions:

- The product can be washed in a washing machine (maximum washing temperature 30 °C, gentle cycle) in a soapy solution or using a special preparation for washing compressing stockings and orthosis using a wash bag
- Do not use fabric softeners
- Do not use centrifugation; squeeze the wet product between two towels
- Dry the product in a horizontal position away from a direct heat source (radiator, sun)
- Do not iron the product
- The maximum number of washing cycles is 120
- Store in a dry and dark place, preferably in the original packaging
- Do not interfere mechanically in the product, do not repair places damaged by use, do not cut the edges of the product
- Pay special attention to the velcro straps:
 - - They may get damaged if in direct contact with the knitted fabric of the product
 - - Prevent their contact with other materials (cloths, textile fibres, dirt) as this may partially or completely degrade their properties and functionality
- The product may not come into contact with organic solvents or bleaching agents

The medical device may be used safely for 36 months from the date of manufacture. This date is contained in the batch number of the device expressed as YY/MM-1234567 where YY is the year and MM the month of manufacture.

Care symbols:

Do not use fabric softeners

Prohibited use:

Stop using the product immediately if:

- If it shows signs of mechanical damage (replace the product)
- Your health has deteriorated which could be associated with the use of the product – consult your doctor at once

Composition:

60 % PAD Nylon, 21 % elastane LYCRA®, 19 % PES Polyester, cotton terry, silicone, metal

This medical device does not contain an active substance, including human blood or plasma derivatives, tissues or cells of human origin or derivatives thereof, tissues or cells of animal origin or derivatives thereof.

Disposal:

The product can be disposed of with normal household waste or in a textile waste container. Although the product is intended for use on healthy and intact skin, it may be contaminated with body fluids during use. In that case, dispose of the product as an infectious material.

The material used is not toxic unless burned. Do not burn the used product.

Any serious adverse event that occurs in connection with the device should be reported to the manufacturer and the competent authority of the Member State in which the user/patient is resident.

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Manufacturer:

ARIES, a.s., Studenec 309, 512 33 Studenec
Last reviewed: 01.03.2022

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- Double face, sangles, coussinets en silicone, tailles XS - 5XLL

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We proudly certify that

Aries A.S.

We certified all necessary tests that the respective articles have successfully passed all tests according to the stringent "Aries" Aries Standard and that it is certified as a regular quality control program.

The company has been approved as "Supplier" license partner and has the right to distinguish its respective articles with the "Aries" quality seal.

INTERNATIONAL CERTIFICATE OF ISO 9001:2015 REGISTERED COMPANY

OEKO-TEX®
CERTIFICATE

The company

Aries, a. s.
Studenec 309
15200 Studenec uhralsky, CZECH REPUBLIC

is production/processing according to STANDARD 100 by OEKO-TEX® for use in the CATEGORY: USE by OEKO-TEX® Standard, American and that report 1909-19012

for the following articles:

Multiple categories and supporting goods (shirts, shirts/blouses, semi-sleeved sport products, sport and leisure trousers, outdoor coats and the like, etc.) and other items, which prove freedom with respect to harmful substances by STANDARD 100, produced from materials already certified according to STANDARD 100 by OEKO-TEX®

Products of the respective main categories (SHIRTS) USE by OEKO-TEX® Standard is certified when they are marked by the OEKO-TEX® green certificate (certification certificate of OEKO-TEX®) and printed with the OEKO-TEX® logo prominently and legibly on the product and the respective label.

This certified product falls completely under Aries Ltd. of OEKO-TEX®. In case of non-compliance, please inform us, we will take the necessary measures to ensure compliance with the OEKO-TEX® Standard and to ensure the product meets the requirements of the OEKO-TEX® Standard.

Issuance of this certificate, which has been a condition precedent according to STANDARD 100 is subject to obligation from the date of issue of this certificate until it is replaced with a new one for another certificate (P/CM) with the exception of the certificate (P/CM) for the following reasons:

This certificate 00046 is valid until 31.03.2018

Since 15.03.2017

OEKO-TEX® Standard 100 cert. reg. no. 1909-19012

FOR YOUR BUSINESS CONTACT US AT: TEL: 0042 73 2222 2333

Aries
MEDICAL PRODUCTS

ARIES, a.s., 512 33 Studenec 309
MADE IN CZECH REPUBLIC
ISO 9001:2015

www.avicenum.eu