# *Avicenum* **ORTHO 1500**

**(EN) INSTRUCTION MANUAL**

Avicenum ORTHO 1500 Knee Orthosis Type 03

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Avicenum ORTHO 1500 Knieorthese Typ 03

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Avicenum ORTHO 1500 ortéza kolenná typ 03

**(EN) Dear customer,**

In your hands, you are now holding a knee orthosis for post-operative and post-traumatic therapy. You can maximize the benefits of the product by its proper use and maintenance.

**Avicenum ORTHO 1500 Knee Orthosis Type 03** is manufactured by the flat knitting technology. The materials used are non-irritating and latex-free. The orthosis is reinforced on the sides with metal supports with a biaxial joint. In the patellar part, the product is completed with a circular silicone pad that slightly compresses the patella and ensures its correct position in the knee joint during movement. In the upper and lower part, the product is supplemented with velcro straps. The mechanism behind the effects of the product is based on the compressive action in the knee area which produces the desired therapeutic effect. In order to achieve the desired effects, you must select the correct size of the product. The product is most effective during physical strain. The product is also suitable for sports activities. You should remove the product during longer breaks. **Avicenum ORTHO 1500 Knee Orthosis Type 03** is a medical device intended for use both at home and in medical facilities.

**Product name:**

**Avicenum ORTHO 1500 Knee Orthosis Type 03**

**Product variants:**

- Double-sided, straps, biaxial joint, silicone pad, sizes XS – 5XL

**Intended use:**

A medical device designed to support the knee during post-traumatic and post-operative therapy, in case of flabby muscles, inflammation, or injury to knee ligaments and tendons and to prevent swelling. This product is also suitable for sports activities as a therapeutic or preventive aid.

**Indications:**

- Acute or chronic knee instability
- Treatment after severe or mild distortions of the knee (sprain)
- Knee post-traumatic swelling
- Patellofemoral pain syndrome
- Patella chondropathy
- Meniscopathy
- Knee swelling in arthritis or arthrosis
- Treatment after surgery or fracture of the knee
- Painful syndromes of the knee
- Prevention of knee injury
- Knee ligaments instability

If this is the first time you use the product, consult your doctor or have your medical condition regularly examined.

**Contraindications:**

- Skin diseases or injuries in the knee area
- Acute weeping skin manifestations
- Hypersensitivity to materials used
- Lymphatic drainage disorders and unspecified swelling of the soft tissue in the knee area
- Limb sensitivity disorders, peripheral neuropathy (e.g., in diabetes mellitus or varicose veins)

Your doctor should consider the use of the product in view of the known contraindications and possible risks compared to the product's therapeutic benefits.

**Target group:**

Patients with knee instability, patients undergoing post-operative or post-traumatic therapy of the knee, and patients with the above indications. The product is intended for use on healthy, intact skin. The product is intended for all age groups and both sexes. The product may be used in pregnancy without limitation.

**Side effects:**

Side effects may occur especially when using a wrong size of the product or if the product is used outside strenuous activities. If the product is too tight, there may be pressure sores, impaired blood circulation due to compression of blood vessels, or deterioration of sensitivity due to excessive compression of the nerves. Sensitive and hypersensitive individuals may experience skin irritation (redness, itching, or blistering) at the point of contact of the product with the skin.

**How to choose the correct size:**

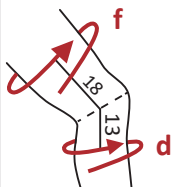
In order to achieve the desired effects, you must select the correct size of the product. Determine the size by measuring the circumference at the points indicated in the figure.

**Note:**

Any swelling or instability of the knee should be examined by your doctor. If this is the first time you experience swelling or instability of the knee, consult your doctor before using the product. Do not use the product if you do not know the cause of the swelling or instability of the knee. Always consult your doctor before combining the product with other medical devices (e.g., compression stockings).

- Use a tape measure.
- Take the measurements standing up, preferably in the morning right after waking up.
- Get assistance from another person.
- Measured values may change during the therapy – always verify the correct size before using a new product.

	Circumference 18 cm above the centre of the knee joint (cm)	Circumference 13 cm below the centre of the knee joint (cm)
XS	38 - 41	28 - 31
S	41 - 44	31 - 34
M	44 - 47	34 - 37
L	47 - 50	37 - 40
XL	50 - 53	40 - 43
XXL	53 - 56	43 - 46
3XL	56 - 59	46 - 49
4XL	59 - 62	49 - 52
5XL	62 - 65	52 - 55

If you are on the border between two sizes, choose the bigger one. If you have difficulties choosing the size of the product, visit a specialized medical supplies store or directly the manufacturer.

If you have difficulties putting on the product, visit a specialized medical supplies store and request a demonstration of the correct method of applying it.

#### With a correctly selected size:

- You should feel noticeable but not unpleasant pressure at the knee area
- The product does not slide or roll down when walking
- The product does not choke at any point

Size and manufacturer information can be found on the tag sewn at the top edge of the product

#### Method of use:

Grab the product by the side metal supports in their upper part and then gradually pull the product over the knee so that the circular silicone pad fits to the patella. Never use force. It is best to put on the product in the morning (while sitting) before any physical strain. When putting on, the product may not come into contact with sharp objects (e.g., jewellery) or long nails.

Stretch the product evenly without creases for effective distribution of pressure. Finish by tightening the velcro straps so that you feel mild pressure.

If you use body cosmetics, wait about 15 minutes after application before you put on the product.

In the case of visible mechanical damage to the product, stop wearing it and preferably replace it with a new product.

### Care and maintenance:

**Avicenum ORTHO 1500 Knee Orthosis Type 03** is a medical device intended for repeated use by a single person. The product will maintain its properties and safety for 12 months from the first use under these conditions:

- Provided you remove the metal supports with a biaxial joint, the product can be washed in a washing machine (maximum washing temperature 30 °C, gentle cycle) in a soapy solution or using a special preparation for washing compressing stockings and orthosis using a wash bag
- Do not use fabric softeners
- Do not use centrifugation; squeeze the wet product between two towels
- Dry the product in a horizontal position away from a direct heat source (radiator, sun)
- Do not iron the product
- The maximum number of washing cycles is 120
- Store in a dry and dark place, preferably in the original packaging
- Do not interfere mechanically in the product, do not repair places damaged by use, do not cut the edges of the product
- Pay special attention to the velcro straps:
  - They may get damaged if in direct contact with the knitted fabric of the product
  - Prevent their contact with other materials (cloths, textile fibres, dirt) as this may partially or completely degrade their properties and functionality
- The product may not come into contact with organic solvents or bleaching agents

The medical device may be used safely for 24 months from the date of manufacture. This date is contained in the batch number of the device expressed as YY/MM-1234567 where YY is the year and MM the month of manufacture.

### Care symbols:

### Prohibited use:

Stop using the product immediately if:

- If it shows signs of mechanical damage (replace the product)
- Your health has deteriorated which could be associated with the use of the product -consult your doctor at once

### Composition:

60 % PAD Nylon, 21 % elastane LYCRA®, 19 % PES Polyester, cotton terry, silicone, metal

This medical device does not contain an active substance, including human blood or plasma derivatives, tissues or cells of human origin or derivatives thereof, tissues or cells of animal origin or derivatives thereof.

**Disposal:**

The product can be disposed of with normal household waste or in a textile waste container. Although the product is intended for use on intact skin, it could have been contaminated with body fluids during use. In that case, dispose of the product as an infectious material.

The material used is not toxic unless burned. Do not burn the used product.

**Any serious adverse event that occurs in connection with the device should be reported to the manufacturer and the competent authority of the Member State in which the user/patient is resident.**

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**Manufacturer:**

ARIES, a.s., Studenec 309, 512 33 Studenec  
Last reviewed: 01.01.2020

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